


Instruction Manual of Pill Box with Dosing Reminder

Features

- Display hour and minute
- Power Saving Alert (PSA), continuously reminding three times every minute for ever unless MODE/CONFIRM key is depressed.
- Totally four reminders a day
- Use 1 pc of LR44 battery

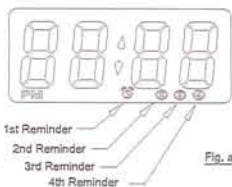

Keys

4. MODE – To enter each reminder mode, to return to real time, or to confirm dose taken, and stop the reminder alert
5. SET – To enter each setting mode of real time or each reminder time
6. ADJ – To adjust reading of real time, and each reminder time, or to enable and disable of each reminder.



How to Set Real Time

4. At real time mode press "SET" key to enter hour setting mode, the reading of "hour" becomes blinking. Use "ADJ" key to adjust it.
5. Press "SET" key again to enter minute setting mode, the reading of minute becomes blinking. Use "ADJ" to adjust it.
6. Press "MODE" again to finish time setting and return to real time.

How to Set Reminder Time

At real time mode, the first press of "MODE" key to enter the 1st reminder. The related symbol appears and flashes. Follow the same way as real time setting mentioned in the above paragraph to adjust the 1st reminder time. Make sure the "Hour" and "Minute" reading are steady displayed and the 1st reminder symbol is blinking. This means the 1st reminder is not enabled yet. Press "ADJ" to enable the reminder. The symbol becomes steady displayed. Press "ADJ" again to disable the reminder. The symbol becomes blinking again. Use "MODE" again to enter the 2nd, 3rd, or 4th reminder, and apply the same method to set each reminder.

How to Know the Reminder is Engaged

At real time, the reminder symbol appears steady means this reminder is engaged. The reminder, which symbol is not appeared, means this reminder is not engaged. At reminder mode, the related symbol blinking means this reminder is not engaged. While the symbol is displayed steady means the reminder is engaged. Use "ADJ" to enable or disable the reminder.

How to Enable or Disable Each Reminder

At real time mode, continuously press "MODE" to find the reminder, which you are going to set enable. If the related symbol is in blinking, press "ADJ" to enable it. The related symbol becomes steady displayed. Then press "MODE" try to return to real time or wait for 10 seconds the display return to real time automatically.


How to Confirm the Reminding Alert

The symbols of all enabled reminders should display steady at real time mode. When the reminder time comes up, it related symbol becomes blinking and reminding alert goes off for three times, then waiting for one minute and repeat again. You don't need to open the cover, just press the red "STOP" to confirm dose is taken, otherwise the reminding alert will never stop. At the meantime the symbol becomes steady again. It will come up again the next day at the same time. Be sure to use the same method to confirm the reminder and stop the alert. If the reminding becomes not necessary, then use "MODE" to enter the reminder, and press "ADJ" to disable the reminder.

How to Replace Battery

One piece of LR44 is used. Please follow the steps to replace battery:

9. Use screwdriver to remove four screws on the backside.
10. Use fingers to remove the front case carefully.
11. Use screwdriver again to remove two screws "A", and then take out battery stopper "B".
12. Remove battery "C" as shown. Discard the old battery.
13. Insert a fresh battery (positive side up) into battery holder "D".
14. Put back battery stopper "B", and screw back the two screws "A".
15. Install back the front case properly. Make sure the buzzer "F" seat on its holder "E" properly.
16. Use driver again to screw back the four screws.

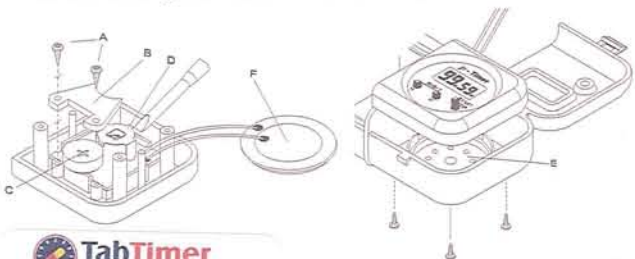

www.TabTimer.com.au

1300 TAB TIMER
(1300 822 846 37)

ABN: 99 137 415 948

Printed in Taiwan