

11. Others

Press button 3 for a few seconds. Step between the settings by pressing Button 1. The settings are made by using buttons 2 or 3.

"BEFORE": How long before the function explained at 10 above will work

"AFTER": How long after the alarm shall be active.

"SOUND": Chose the sound. Four different are available.

"LIGHT": Chose Light on/off

"HOURS": Chose between 24 and 12 hours .

"RECORD": The 28 latest takings of medicine are displayed. Latest=1.

"DATE" Set year, month, day. The time will automatically change between summer/wintertime. If year (YY) = 0 the time will not change

"TEST": Press button 3: The tray turns and the alarm sounds.
Press button 2: This shows how "BEFORE" is working.

Tip: If one compartment is not big enough to hold one dose, use two compartments and set the same alarm time for both.
If the Careousel is lying upside down the next dose will not be available until the unit is turned right.

Maintenance: Wipe off the Careousel with a damp cloth. The tray could be washed by hand. Exchange the batteries at least once a year. Four Alkaline batteries should be used.

Security: If the tray does not move as it should, a signal sounds and "ERROR" is displayed. Investigate the reason.
If the Careousel has been lying upside-down during an alarm Sequence, a warning signal sounds. Press button 3 to shut off. Place the tray in the right position.

Comm.: The Careousel has a flexible communication interface. Connection can be made to other devices such as alarm lights, modems, wireless transmitters etc.

Accessories: Key, prescription card to put under the Careousel, Instruction for the lid and 3 day/time discs are enclosed.
Extra tray with lid and flash-light are available.
The accessories could be ordered at www.pharmacell.se.

Instructions for Careousel®

The right medicine at the right time



TabTimer
Hjälper dig att ta dina mediciner på rätt tid

www.TabTimer.com.au

1300 TAB TIMER
(1300 822 846 37)
ABN: 99 137 415 948

1. Activate the batteries

Take away the battery lid. Pull the red stripe. This is the first time the Careousel is used.




2. Open the lid

Open the lid by pressing the front at the opening. If your Careousel is used, it could be locked. See No 9 to unlock.



3. Set the clock

Press button 1 till "CLOCK" is visible on the display. Use button 2 or 3 to set the clock.

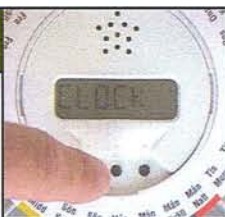

4. Chose No of doses

Press button 1. "DOSES" are visible on the display. Wait a few seconds. 01, 02 etc is shown. Step with button 2 or 3 until the right no of doses is shown in the display. Up to 28 Doses per day could be set.



5. Set alarm times


Press button 1 until "ALARMS" is shown. Wait a few econds. The time for the first alarm is shown. Set desired alarm time with button 2 or 3. If more doses — set next alarm by pressing button 1.



2


6. Day-time discs

Place the appropriate disc for the right number of doses. There are discs for 1,2 3 or 4 doses per day. For more doses there is an empty disc to write on. Dispense the medicine in the tray.

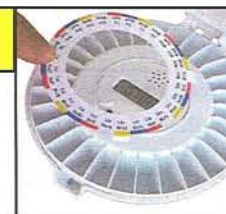

7. Place the tray right

Place the tray so that the next dose is placed next to the right of the opening. The tray is separate and could be exchanged to another filled tray. An extra tray and lid can be obtained.

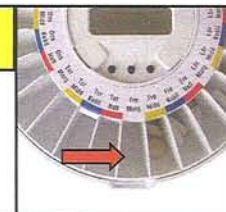

8. Take the medicine

At the pre-programmed times, the dispenser rotates, the alarm signal is heard, and the correct dosage comes into view through the opening in the lid. The alarm continues to ring for up to 60 minutes.



9. Lock

The Careousel could be locked. The lock is under the Opening. Twist carefully 1/4 turn.



10. Take the medicine before preset

It is possible to take the medicine before the alarm time by turning the Careousel upside down two times a while before the preset alarm time. When it is turned right again the Careousel moves one step and the next dose is available.



3