

The VitaCarry[®] Advanced Medication/Pill Reminder




Advanced Pill Reminder
Simplifies complex dosing
schedules.

**Care and
Use Guide**

Features

Congratulations on the purchase of your new electronic Timer. With proper care and use, you should expect years of reliable service. Please familiarize yourself with its' many useful features and read through the following instructions before you begin to use it.

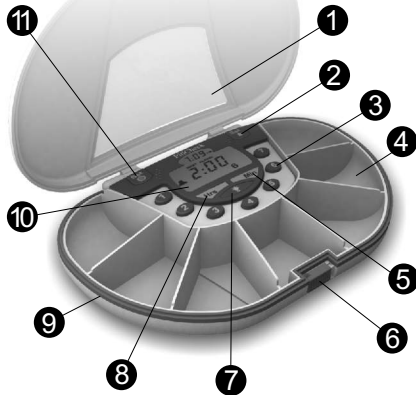

1. Viewing lens
2. SET ALARM key
3. NUMBER key
4. Compartment
5. MINUTES key
6. Release Latch
- * 7. BACK LITE key
8. HOURS key
9. VitaCarry®
10. DISPLAY
11. SET TIME key


* Available on some models.

BEFORE YOU BEGIN





- Read all instructions.
- This item is designed for use by adults only.
- Keep medications/supplements away from children.
- Do not use Timer accessory near water.
- Do not use for other than intended purpose.
- Save these instructions.

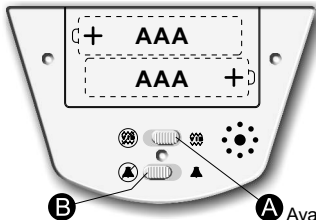Install **AAA Batteries** in back of Timer as shown. You are now ready select initial settings.

GETTING STARTED



To set Vibrate and Audio

1. Slide **(A)** to  or  to turn Vibrate alarm on or off.
2. Slide **(B)** to  or  to turn audible alarm on or off.

(A) Available on some models.

SET TIME

To Set Time

1. Press **(A)** and **SET TIME** will appear on the display.
2. Choose **(B)** Hrs & Min.
3. Press **(A)** again to confirm.

Note: If the AM indicator is not on, it is assumed that the time is PM. The same is true for the Alarm time.



Back Lite: Press once to enhance display for up to 16 seconds.

SET ALARM

To Set Alarm

1. Press **(C)** and **SET ALARM** will appear on the display. All enabled alarms will be displayed and the lowest numbered alarm will begin blinking.

Note: The blinking number means you may change that Alarm Time using the Hrs & Min keys. Pressing its' Number Key will disable it.

2. For each alarm you wish to set, press a Number Key and set the Alarm Time by using the Hrs & Min keys.

Note: Min keys move in 10 Min intervals to speed up the process.

3. Press **(C)** to confirm your Alarm times and exit Set Alarm mode.

ALARM USE

When an Alarm goes off it will blink and the Sound and Vibrator may go on. If Alarm is not acknowledged, Sound and Vibrator will turn off after 10 minutes. Alarm will still Blink. If Alarm is still not acknowledged after 1 hour, it will be marked with a X. This means that you have missed the time you were supposed to take the Pill.

*see Alarm Use continued
on reverse side.*

ALARM USE

continued from other side.

After a Missed pill, the Next Alarm will be displayed and the Missed Alarm number is displayed with the X through the number.

After an Alarm is acknowledged, the Next Alarm is displayed. If there are no more Alarms for the day, 0:00 Alarm Time will be displayed.

At midnight, 1st Alarm of the day will be displayed. All Missed pills are reset.

HINT: Press any Number Key for a quick view of Alarm Time that key is set for.

Display at a glance

Time: It is 8:32 AM.



Missed Pill:

Compartment 1 was not acknowledged on time and is displayed as a Missed Pill.

Next Alarm:

At 10:00 AM the Vibrate and Audible Alarm will activate and remind me to take pills from compartment 3.

Acknowledge:

Pressing the number 3 turns off the alarm and tells the device to display the Next Alarm (if any)..