

# Automatic Pill Reminder

with advanced features



## Operating Instructions

Advanced Pill Reminder  
Designed to help simplify  
complex dosing schedules.

Model No. GVC-004W

**EG+** ENNOVEA  
LIFESTYLE DEFINED.

# Automatic Pill Reminder

with advanced features

## Operating Instructions



## WARNING

To reduce the risk of injury, everyone using this device must read and understand these instructions before operation. This item is for adult use only. Keep out of reach of children - this product is not child resistant. Do not use for other than the intended purpose. Always consult a qualified professional for any health related diagnosis and recommended treatment. Save these instructions.

Congratulations on the purchase of your new **Automatic Pill Reminder**. Please familiarize yourself with its many useful features and read through the following instructions before you begin to use it.

### Recommended Use

As an aid to improve medication/supplement adherence, we designed this system to be simple to use and convenient to carry and keep nearby.

Your Automatic Pill Reminder can help you manage up to four scheduled doses per day. As an added convenience, the Pill Reminder can be removed from its case and carried with you throughout the day.

**NOTE:** This product is not child resistant and should be kept away from children.

# Set Up the Pill Reminder


---

The **Advanced Pill Reminder** is easy to set up and use. From the back, remove the Battery Cover and install **2 AAA Batteries** (*not included*) in the **Battery Compartment** as shown.

Slide the **Vibrate On/Off Switch** to select the desired Alarm feature.

Slide the **Audio On/Off Switch** to select the desired Alarm feature.

Battery Compartment

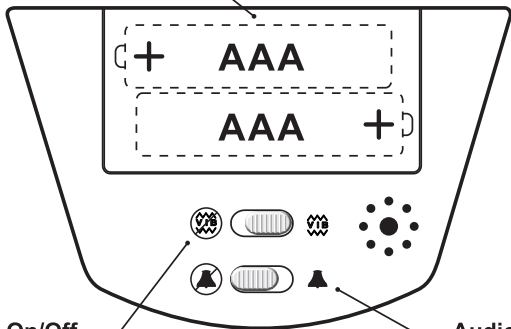

Vibrate On/Off

Audio On/Off

Back View

# To Set Time


---

To set the **Time of Day**, press **Set Time** once.

Press **Set Hours** and **Set Minutes** until the desired **Time of Day** is shown on the display.

Press **Set Time** again to confirm.

**NOTE:** The display does not indicate PM. To set for PM, be sure that AM is NOT displayed.

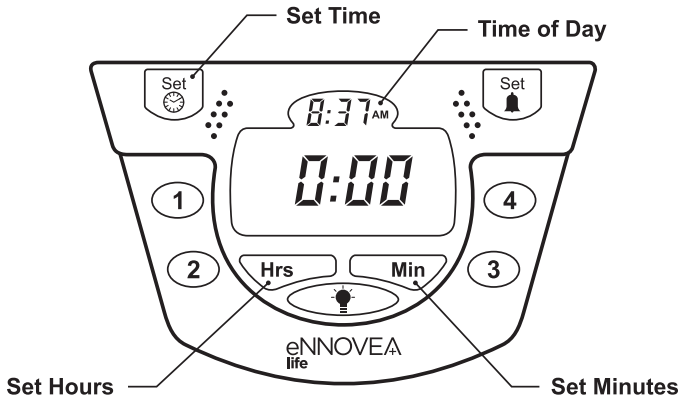

Front View

# To Set Alarms

---


To set an **Alarm**, press **Set Alarm** once.



Press the **Alarm Button** for the Alarm you wish to establish. (e.g. Alarm Button 1)

Press **Set Hours** and **Set Minutes** to the desired **Alarm Time**.

Press **Set Alarm** again to confirm. Repeat the process to establish your other Alarms.

**NOTE:** The display does not indicate PM. To set for PM, be sure that AM is NOT displayed. Also, Alarm Times advance in 10 minute increments.

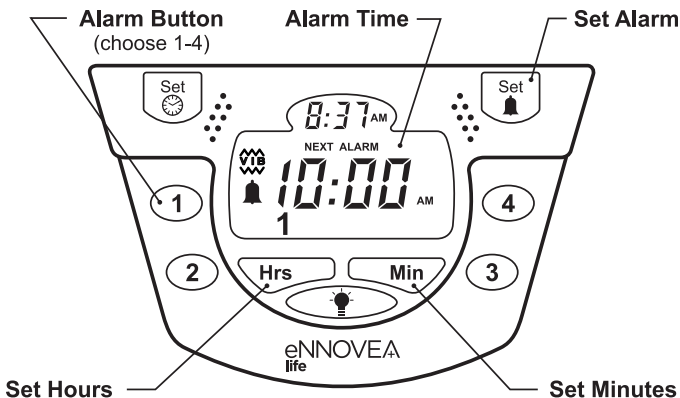

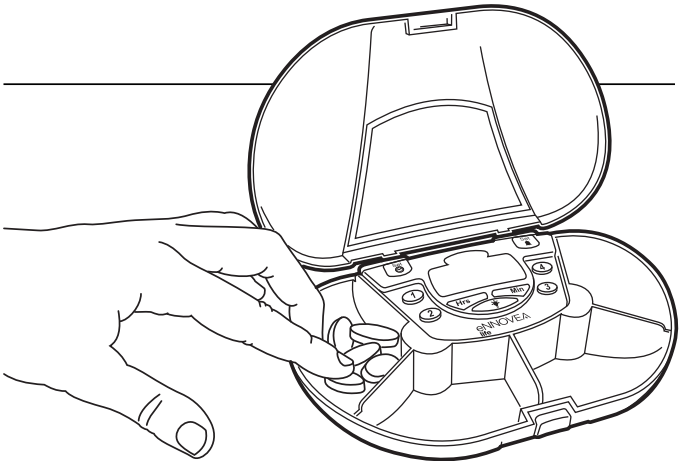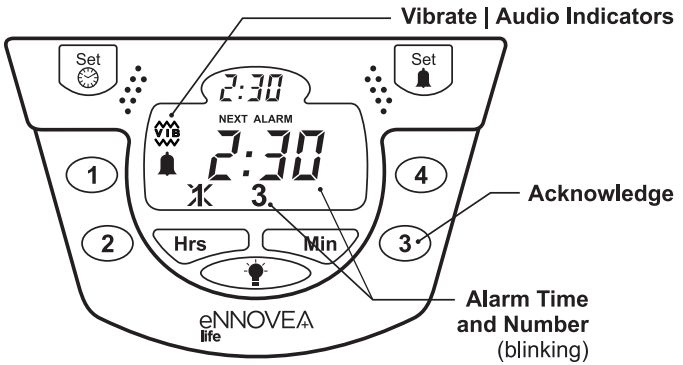

# Using your Pill Reminder

---

With **Time of Day** and **Alarm Times** scheduled, your device is ready. The **Time of Day** is **2:30 PM**. (notice there is no AM displayed)  
The **NEXT ALARM** is also **2:30 PM**. (notice there is no AM displayed)  
Therefore, the Alarm is going off now - the **Alarm Time** and **Number** are blinking together. In this case, it is **Alarm 3**. The device is also Vibrating and Beeping. (notice the **Vibrate | Audio Indicators** are on)  
Press **Alarm Button 3** to **Acknowledge** and take the pills from **Compartment 3** in your **Daily Pill Organizer**.

**NOTE:** Alarm 1 was missed, as indicated by the “X”

---

Press any **Alarm Button** once for an instant display of that **Alarm Time**. **Alarm Times** advance in 10 minute intervals to speed up process.  
**To set an Alarm to ZERO**, press the Alarm Button twice during Alarm set up process.

If an Alarm is not acknowledged in 10 minutes, Vibrate and Audio will stop. The display will continue to blink for one hour. If it is still not acknowledged, the Alarm Number will show an “X” through it - indicating a **Missed Dose**.

When all the Alarms have expired or been acknowledged for the day, **0:00 Alarm Time** will be displayed - indicating no more doses are scheduled for that day.

At midnight, the first Alarm of the day will be displayed. Any missed doses are reset.