

automatic pill dispenser user manual




TabTimer.
helps keep medications on time

www.TabTimer.com.au

1300 TAB TIMER (1300 822 846)

ABN: 99 137 415 948

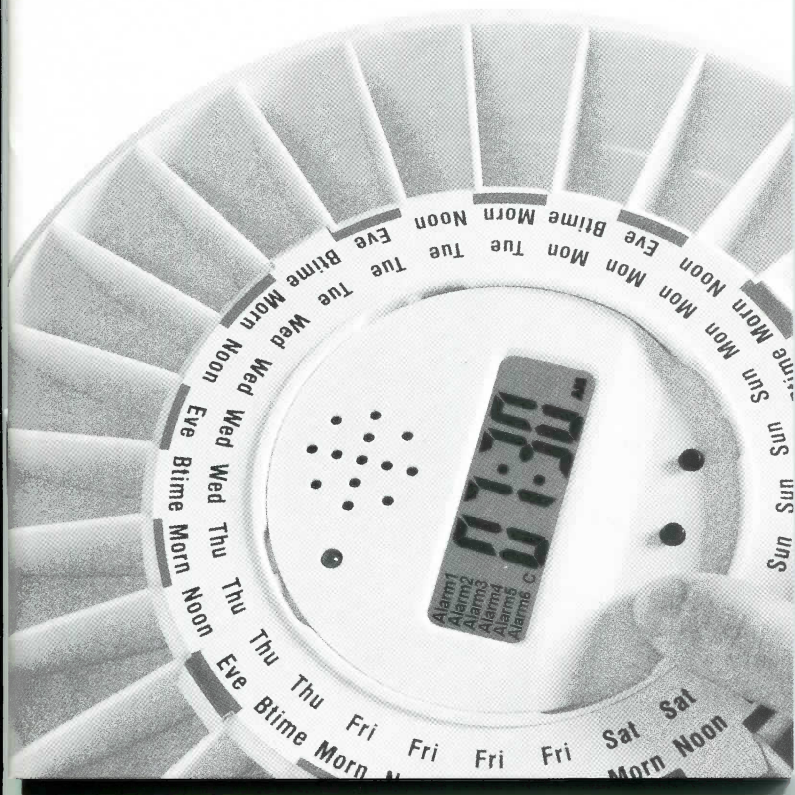

Introduction

Thank you for purchasing our Automatic Pill Dispenser and welcome to the Ivation™ family. This user manual offers you a brief walkthrough of your new device's features and usage protocols. For additional information and technical support, please contact our help team at info@myivation.com.

Intended Use


Whether you're using it to keep track of your own medication or to assist an aging or infirm family member, this pill dispenser promotes better health by alerting users and dispensing their medication at the right times of the day and week. It is also designed with user safety in mind, preventing double dosing, overdosing, and dosage mix-ups.

Box Contents

- 1 x Pill Dispenser
- 1 x Removable Tray
- 6 x Dosage Template
- 1 x Custom Key
- 1 x Owner's Manual

Parts & Controls

- 1. Tray Cover
- 2. Dispense Opening
- 3. Dosage Window
- 4. Cover Release Tab
- 5. Alarm Speaker
- 6. Alert Indicator
- 7. LCD Display
- 8. Button 1
- 9. Button 2
- 10. Button 3
- 11. Removable Tray
- 12. Cover Lock
- 13. Battery Door
- 14. Door Release Tab

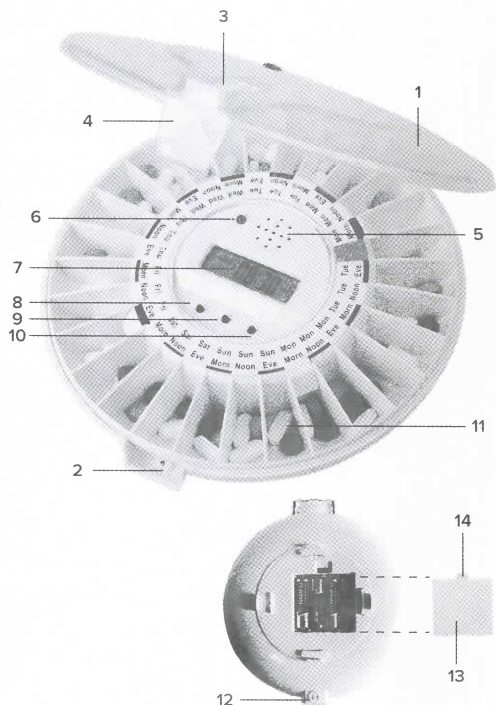


Inserting or Replacing Batteries

- 1 Depress Door Release Tab (14) and remove Battery Door (13).
- 2 If necessary, remove spent batteries and discard according to local code.
- 3 Insert 4 fresh AA batteries in correct polarity and replace door. LCD Display (7) will turn on.


Before First Use

- 1 Depress Cover Release Tab (4) and lift Clear Tray Cover (1) upwards.
- 2 Remove and discard any packaging covering Removable Tray (11).
- 3 Clean tray using a soft damp cloth or clean sponge only.

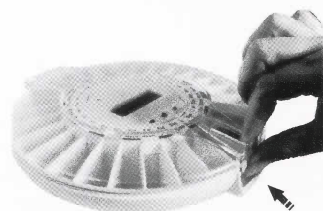


Loading Your Dispenser


Provided with your purchase are 6 dosage templates that simplify loading the dispenser correctly. They identify the day of the week and the appropriate number of doses per day. Select from the following templates to match your medication schedule:

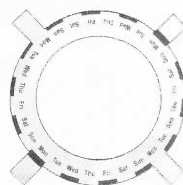


Template 1
one alert per day

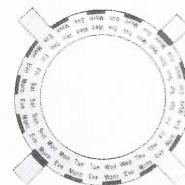


Template 2
two alerts per day

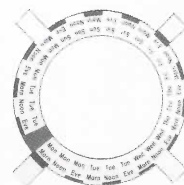


Template 3
three alerts per day

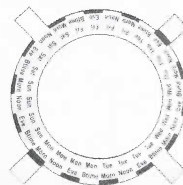

Template 4
four alerts per day

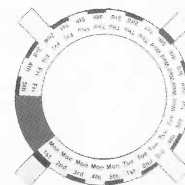


Template 5
five alerts per day

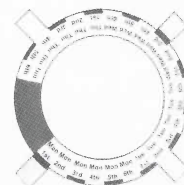

Template 6
six alerts per day

- 1 Set correct template on center of tray and fold tabs down to secure in place.
- 2 Using template as guide, place pills in correct dosages in each compartment.
- 3 Do not place pills in compartments corresponding to red markings on template.

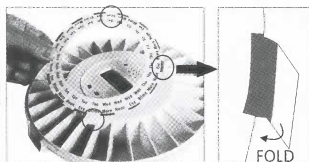

Note: Organizing medication correctly, assigning dose times and determining what medications go into each slot is the responsibility of you, your caregiver, and/or your health care provider.

Remember that refill frequency will depend upon the number of daily dosage:

- A one-a-day regimen will require a **refill every 28 days**.
- A two-a-day regimen will require a **refill every 14 days**.
- A three-a-day regimen will require a **refill every 9 days**.
- A four-a-day regimen will require a **refill every 7 days**.
- A five-a-day regimen will require a **refill every 5 days**.
- A six-a-day regimen will require a **refill every 4 days**.

Setting the Clock


Note! Pill dispenser follows 24-hour time format by default. To switch to 12-hour format, press and hold Button 2 (9) and Button 3 (10) simultaneously for 3 seconds. AM or PM symbol will appear on right side of display when clock is in 12-hour format.




- 1 Press and hold Button 1 (8) for approximately 5 seconds until ":" symbol flashes.
- 2 Press Button 2 (9) to adjust hour digits. Press Button 3 (10) to adjust minute digits. *Note! Pressing and holding buttons 2 or 3 will cause digits to speed up.*
- 3 Confirm clock settings by pressing Button 1 (8).



▲Press button 1

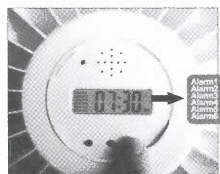

▲Press button 3


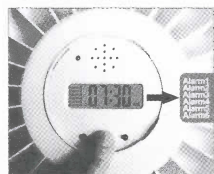


▲Press button 2



Setting & Checking Alarm Times

Your new pill dispenser can store as many as 6 alarms per 24-hour period. Once you set these alarms, they will sound at the same times every day until you reset them.

- 1 Press Button 1 (8) once. "Alarm 1" will flash.
- 2 Press Button 2 (9) to adjust hour digits. Press Button 3 (10) to adjust minute digits. *Note! Pressing and holding buttons 2 or 3 will cause digits to speed up.*
- 3 To cancel a previously set alarm, continue pressing Button 2 (9) until "---" is displayed. This occurs after 23:00 or 11:00 PM.

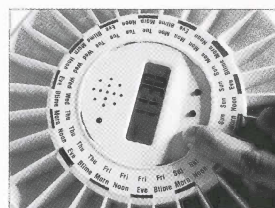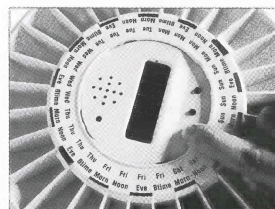

- 4 After setting desired alarm time, confirm by pressing Button 1 (8). "Alarm 2" will then flash.
- 5 Repeat steps 2 through 4 to set Alarm 2. "Alarm 3" will then flash.
- 6 Repeat steps 2 through 4 to set Alarm 3, 4, 5, and 6.
- 7 After confirming Alarm 6, display will return to clock mode.

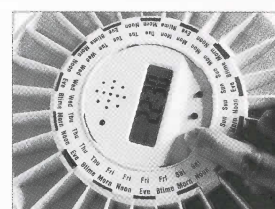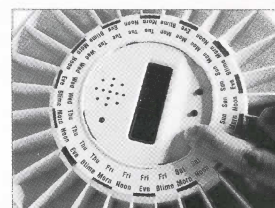

Setting Alarm Tone



Your new pill dispenser features 3 different alarm tones as well as a silent (light alert only) option to accommodate users with different hearing levels. At the top of the display, symbols "A", "B", and "C" indicate the selected tone. When set to silent mode, none of the three symbols will be displayed.

- 1 To change alarm tone, begin by pressing and holding Button 3 (10).
- 2 While holding down Button 3 (10), press and hold Button 1 (8) as well.
- 3 Alarm tones will sound for a few seconds each, allowing you to choose.
- 4 To select a particular tone, simply release all buttons when that tone sounds.




Testing Your Dispenser


- 1 Press and hold Button 2 (9) for 5 seconds. "CH" will be displayed. Device is now in testing mode.
- 2 Press Button 1 (8), and device will automatically proceed through following tests:
 - a. "CH 1" will be displayed, and all display characters will then light up.

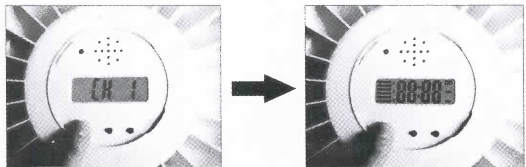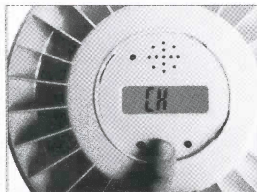

- b. "CH 2" will be displayed, and tray will then rotate one compartment.

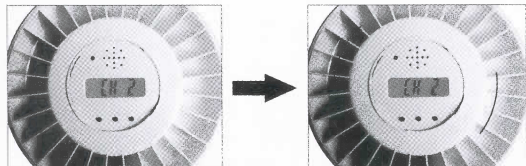

- c. "CH 3" will be displayed, and alarm will then sound and red light will flash.


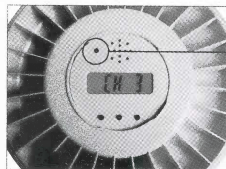


The light becomes red

- 3 Lift dispenser and turn it over as if pouring medication into palm. Alarm and red light will shut.
- 4 To exit test mode and return to clock mode, press Button 3 (10).
Note! If dispenser is not turned over within 10 seconds, "CH 4" will be displayed. If tray does not rotate properly, motor will stop and "CH 6" error message will be displayed. In event of test malfunction, reset by removing and replacing batteries.

Locking Your Dispenser

- 1 Locate Cover Lock (12) on bottom of unit.
- 2 Insert included key and turn counterclockwise.
- 3 To unlock, insert key and turn clockwise.

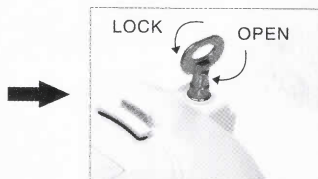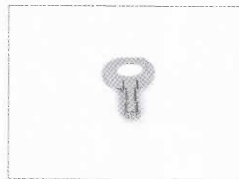

Important Notes

Your pill dispenser is equipped with a special sensor. To **stop the alarm** and dispense the pills, lift the dispenser and turn it over to pour medication into your palm or a suitable container. The alarm and red light will then shut automatically.

A battery symbol with an "X" over it indicates that the **batteries are low** and should be replaced to ensure proper function. To save power, pill dispenser will stand by and shut down if no operation in several minutes. To wake it up, tilt up or press the Button 3 (10).

Do not place your pill dispenser in **direct sunlight** or strong ultraviolet light. It may cause operational malfunction.



TabTimer.
helps keep medications on time

www.TabTimer.com.au

1300 TAB TIMER (1300 822 846)

ABN: 59 137 415 948

Made in China
ivation is a trademark of
C&A IP Holdings, LLC
©2014 All right reserved