



MODEL ILY450 & ILY451

Your easy to set, Global ILY Clock, gives you the convenience of an effective “wake up call” whether you have access to electricity or not. You can operate the alarm(s) by battery power or by electricity using a 4.5VDC 400mA power supply.

To conserve battery life and for maximum results, we suggest using the power supply when electricity is available.

To connect the power supply, plug the mini barrel plug into the jack marked DC4.5V (on the side of the unit) and plug the other end into the electrical. When batteries are installed properly, you will have the advantage of battery back up for your alarm(s) in the event of a power failure.

Slide open the battery compartment cover (at the bottom of the clock) and install four “AA” standard Alkaline batteries (purchased separately). Ensure that the batteries are positioned correctly as shown. Replace the cover. The clock time gets its power from one “AA” battery (located at the top) as shown on the diagram. This battery must be in place and functional for clock time to work. The alarm function(s) require all batteries to be in place when not powered by the 4.5VDC power supply.

BEFORE using your new ILY Clock, take a few minutes to read the following instructions. Your clock offers you variable tone and variable volume for the audible alarm, as well as a Bedshaker (Vibrator) option.

Setting The Time:

At the back of the clock, locate the knob marked with clock hands symbol and turn it in the direction shown by the arrow until the clock hands show the correct time of day.

Setting The Alarm:

At the back of the clock, locate the knob marked with bell alarm symbol and turn it in the direction shown by the arrow until the alarm hand shows the time you want to awaken.

By positioning halfway between the hour markings, your alarm will activate on the half hour; one quarter will give you 15 minutes etc.

ALARM switch must be in ON position for alarm to activate.

Selection of Alarm Mode: (switches located on side of clock)

Audible Alarm: Position switch to AUDIBLE. Rotate Volume Control (marked VOL) to desired loudness level. Rotate Tone Control (marked TONE) from high to low to achieve maximum results for you.

Vibration Alarm: Position switch to VIBRATE. Plug 4.5VDC ILY Bedshaker Vibrator (purchased separately) into jack marked VIBRATOR (on the side of the unit) and place Bedshaker under pillow or mattress. (For pillow use, we suggest putting inside pillow case) The audible alarm will not sound. This is excellent when you do not wish to disturb other household members.

Both Audible & Vibrate Alarms: Position switch to BOTH. Ensure the 4.5VDC Bedshaker (vibrator) is plugged into the clock and that the tone and volume is set to your desired levels.

Snooze Feature (Repeat Alarm)

After the alarm is activated, press the snooze/light button, at the front of the clock. This will stop the alarm(s) in a few seconds. In approximately 4 minutes, the alarm(s) will start again automatically. You can repeat this feature as often as you wish.

Night Time Illumination:

To view the time in darkness, press and hold the snooze/light button. The night light will illuminate.

Service:

Your clock should be serviced by qualified service personnel. The user should not attempt to service the unit.

All servicing should be referred to your distributor or:

Global Assistive Devices, Inc.
1121 East Commercial Boulevard
Fort Lauderdale Florida USA 33334

One Year Warranty

Your ILY Clock is warranted to be free from defects in material or workmanship for one year from date of purchase.

This warranty does not apply to any unit which has been subject to misuse, negligence or accident.

At its option, GLOBAL ASSISTIVE DEVICES, INC. may select to replace the entire unit rather than repair it.

During the warranty period, return the clock, with proof of purchase date, a detailed description of the reason for return, in a secure package, insured and with shipping prepaid. Enclose payment in the amount of \$9.95 for domestic shipping and handling.

Other Countries please contact us for pricing. We accept check, money order, Visa, MasterCard, Discover, or American Express.