



TabTimer
helps keep medications on time

www.TabTimer.com.au

1300 TAB TIMER
(1300 822 846 37)
ABN: 99 137 415 948

Four Alarms Pill Box

FEATURES:

As many as four different times can be alarmed each day. Each alarm will repeat every 24 hours. The alarm will sound for 60 seconds and unless it is turned off (by hitting any key), it will sound a reminder every minute for half an hour.

BUTTONS:

ALM Switches *ALARM* on or off for each alarm time you have set.

ADJ Advances the *HOURL* and *MINUTE* settings for both the clock and the alarm times.

SET Selects the *HOURL* or *MINUTE* for both the time and alarm settings.

SETTING THE TIME AND ALARMS:

TIME SETTING

The first press of the SET button will select the HOUR of the clock. Press the ADJ button until the desired hour is displayed. Be sure to note that the clock is a 24-hour clock.

The second press of the SET button will select the MINUTE of the clock. Press the ADJ button until the desired minute is displayed.

ALARM SETTING

The alarm settings are selected similarly to the time settings. Pressing the SET button first selects the time settings. Continuing to press the SET button selects the alarm settings. At the bottom of the display screen it will indicate which alarm is in the adjustment mode. Set the hour and minute to the desired alarm times. Once time is set, then use the ALM button to turn the alarm on or off, a bell on the display screen will display when the alarm is on. If you are setting a PM time, a P will be displayed on the left hand side of the display.

TURNING ALARMS ON OR OFF

The alarms are switched *ON* or *OFF* by pressing the ALM button while the specific alarm is displayed on the screen. When the alarm is set, a small bell is displayed on the screen. The alarms may be switched on or off when the alarm times are initially set or they may be switched any time later on.

Any number of alarms may be used. Only alarms that are set to beep will be used. Unused alarms will not affect the operation of the timer.