

PILL REMINDER USER MANUAL

1. Appearance

2. Function of keys

STOP Key:

- (1) To shut off the beeping alarm during alarm cycle.

SET Key:


- (1) Hold this key down for 3 seconds to enter Time Setting Mode.
- (2) In Time Setting Mode, switch between Hour, Minute, and Normal Mode.

ADJ Key:

- (1) To toggle 12/24- hour clock.
- (2) To set the clock in Time Setting Mode.

RESET Key:

If errors are displayed on the LCD or an unusual beeping alarm sound occurs (caused by changing batteries, etc), press the RESET key to return to the normal condition.

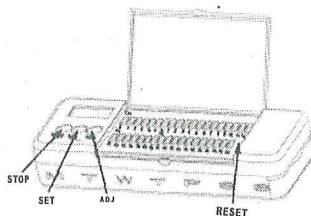

3. Operation

(1) Setting the Correct Time

Press the SET key for 3 seconds to enter the Hour Setting Mode (hour blinking).

Press the ADJ key to set the hour, then press the SET key again to enter Minute Setting Mode (minute blinking)

Press the ADJ key to set the minute. Press the SET key to return to Normal Mode.

In the Time Setting Mode, you can switch between Hour, Minute, and Normal Mode just by pressing the SET key.

If one minute elapses in Time Setting Mode without any key being pressed, it will return to normal mode automatically.

(2) Toggling 12/24-hour Clock

In Normal Mode, press the ADJ key to switch the time system from the 24-hour clock to the 12-hour clock, and vice versa.

(3) Setting Alarm as a Reminder to Take Medicine

To set the alarm as a reminder to take medicine, slide the alarm switch corresponding to the desired alarm time to the up "on" position. From 6:00, 6:30.....to 12p.m., there are 37 preset time slots which can be selected as the alarm time to remind you to take your medicine. If an "alarm clock" is turned on, the pill reminder will beep to remind you at the appropriate time. If multiple "alarm clocks" are set, they will sound one by one to remind the user.

(4) Shutting Off Beeping Alarm "MISSED PILL?" Display

You can shut off the beeping alarm by pressing the STOP key. Otherwise, it will toggle between 10 seconds of beeping followed by 10 seconds of silence for 4 minutes until stopping automatically. If the STOP Key is not pushed during the 4 minute alarm cycle, "MISSED PILL?" will flash on the LCD display. Pressing the STOP Key will remove the flashing "MISSED PILL?" from LCD display. THE "MISSED PILL?" INDICATOR DOES NOT NECESSARILY MEAN THE USER DID NOT TAKE THEIR PILL. It is a reminder that the pill MAY NOT have been taken because the normal process of pressing the STOP Key was not accomplished during the 4 minute alarm cycle. Any time pills are missed, you should follow the directions from your doctor before taking a pill out of sequence. Some medications require you to wait until the next scheduled period to take your pills and others can be taken right away...YOU SHOULD ALWAYS CONSULT WITH YOUR DOCTOR IF YOU MISS TAKING YOUR MEDICATION ON SCHEDULE.

(5) Changing the Batteries

After several months of use, you will hear a single beep once every 15 minutes indicating low battery voltage.

The low battery voltage indicator icon will also appear on the LCD display. In this situation, you should replace the batteries without delay. Once the batteries are replaced and the battery compartment cover is closed, reset the correct time. Your preset "alarm clocks" will continue to operate as normal.

*Please Note:

Although the Pill Reminder unit is built to high quality standards, it is nevertheless a mechanical device that relies on battery power and may in some instances fail to sound an alarm. It is intended only as an audio aid to help those that may need an occasional reminder to take their medication. You are solely responsible for the proper timing and dosage of your medication. This device should not be used by those who suffer from any ailment that may cause them to routinely forget to take their medicine or those who are not able to understand the proper use and intent of this device as an additional, secondary reminder aid.