

TABTIME SUPER 8 INSTRUCTIONS

www.tabtimer.com.au

- TABTIME SUPER 8 can alarm up to 8 times per day at the exact times you choose and remembers your alarm settings every day.
- TTS8 has 8 tablet compartments numbered to correspond with each alarm.
- There is also a Countdown function for regular doses. i.e. every ½ - 1½ - 2¼ - 3¾ hours etc
- TTS8 has a back up safety feature to ensure you remember to take your medication. The flashing red LED on the lid of TTS8 begins to flash at the same time as the audio alarm & will continue to flash until you open the box to take your tablet. So if the LED is flashing it indicates that you have not taken your last dose of medication. Please note that extended use of this feature will reduce the life of the battery.
- TTS8 has a HI/LO volume switch. Choose whichever volume suits you the best.

TABTIME SUPER 8 has 10 functions controlled by 3 control buttons.

Pressing the MODE button once allows you to set the TIME (24hr clock – military style)

Press Mode 2-3-4-5-6-7-8-9 times to set up to 8 ALARMS

Press 10 times to set the COUNTDOWN function for countdown alarm settings.

Note- the Countdown function cannot be used in conjunction with the variable time alarms
If you make a mistake during programming, you must return to function 1 and start over again.
TABTIME SUPER 8 has a 24 hour clock (24hr clock – military style) and all settings start in the a.m.

TO SET THE TIME. (FUNCTION 1)

REMOVE THE PLASTIC INSULATING STRIP TO CONNECT THE BATTERY –(TYPE CR 2032)

Press and hold the MODE button until the display flashes. Set the TIME using the HOUR & MINUTE buttons. Confirm the TIME by pressing the MODE button to set it. The TIME is now displayed.

TO SET UP TO 8 ALARMS (FUNCTIONS 2-9)

Press the MODE button and a -:- display and number 1 with a bell appears at the top of the screen. Press and hold the MODE button until the -:- display flashes then using the HOUR & MINUTE buttons set the time you require for the first alarm. Press MODE to confirm and set alarm 1.

Press MODE button to set alarms 2-3-4-5-6-7-8 by repeating this process.

At alarm time, the audible alarm will beep for 40 seconds and the red LED will begin to flash.

Both the alarm and flashing light are cancelled by opening the box.

(You can also press the MINUTE button to stop the alarm beeping)

The flashing LED on the lid of Super 8 will continue to flash until you open the box to take your tablet. So if you take the Tab Time Super 8 out of your pocket or purse and find that the LED is flashing, you have forgotten to take your last dose!

TO SET THE COUNTDOWN TIMER (FUNCTION 10)

From the clock real time, press the MODE button 10 times. You will now see an egg timer symbol on the top left of the screen and the display shows 00:00. Press MODE until the symbol 00:00 flashes then set the time you require for the interval between doses i.e. 1¼ - 2½ - 3¾ hours etc. Press the MINUTE button to start the Countdown and the egg timer symbol will begin to flash to confirm the Countdown timer is operating. You can leave the countdown on this screen, or if you like press the MODE button to return to Real Time and the beeper will alarm at the interval you chose. After the first alarm if you want to repeat, go back to the countdown mode and press the MINUTE button again to start the next Countdown sequence.

CANCELLING AN ALARM

To delete any alarm; Press the MODE button to go to the alarm 1-8 that you wish to delete and press the MINUTE button to delete that alarm. To change all settings take the battery out for a few minutes.

TROUBLESHOOTING.

Check the battery power indicator in the top right hand corner is at least 2 bars. If your Super 8 starts to act strangely, the figures fade or the Alarm rings randomly etc it is time to change the battery which can be bought at TabTimer, most supermarkets, watch repairers, or jewellers. If you find the catches on the tablet compartments hard to open, file a little off each catch with a nail file.