

SIMPLE INSTRUCTIONS TT4-28 – Automated Pill Dispenser

4 alarm model



FUNCTIONS OF THE TIMER

- TIME – The current time is shown on the device
- ALARM (up to 4 per day) - Repeats at same times each day without reprogramming
- DURATION – Alarm sounds for 30 minutes until tipped to access pills
- EASY STOP - simply tip the device to access the pills
Then the device will automatically sound at the next programmed time



QUICK DEMO

1. Use the built in test function to demonstrate the rotation and alarm
 - a. Press button '1' for approximately 12 seconds, until 'CH' is displayed
 - b. Press button '2' to run through the test
 - c. The device will show all display, rotate, alarm and flash
 - d. Tip the device to stop within 10 seconds or wait for it to return to CH
 - e. Repeat the test by pressing button '2'
 - f. When finished exit the test mode by pressing button '3'

TO CHANGE THE ALARM SOUND/TONE

1. Hold down button '3' and at the same time hold button '1'
2. The device will rotate around 3 different alarm tones and also a silent setting
3. Symbols indicate the chosen tone ;
 - A - Tone 1
 - B - Tone 2
 - A+B - Tone 3
 - Blank - No audible alarm, just flashing light
4. Release the buttons when the desired tone is heard

TO SELECT 12 OR 24 HOUR FORMAT

1. Press buttons '2' & '3' simultaneously to switch between 12 hour or 24 hour display

TO SET THE TIME

1. Press button '1' for approximately 6 seconds, until the colon symbol (':') flashes
2. Press button '2' to change the hours, Press button '3' to change the minutes
3. Press '1' to lock in the time setting

TO SET THE ALARM TIMES

1. Press button '1' and release
2. 'Alarm 1' symbol should be shown, flashing on the left hand side
4. Press button '2' to change the hours, Press button '3' to change the minutes
5. Press '1' to lock in the time setting and advance to the next alarm
6. 'Alarm 2' symbol will flash, continue as above the change the hours & minutes
7. Continue as above to set alarms 3 & 4 if required and press 1 to return to the time display.

TabTimer™ helps keep medications on time™ | www.TabTimer.com.au | 1300 TAB TIMER (1300 822 846)

Pricing and specifications are correct at the time of publishing and may change without notice. Your health professional should be consulted prior to using any TabTimer product to establish the suitability of a TabTimer product for use, administration or storage of any medications. In purchasing and continuing to use our products you have agreed to and accept the Terms and Conditions shown on our website. Images are for illustration purposes only. Errors and omissions excluded.

TabTimer™, 'helps keep medications on time™' and the TabTimer™ logo are registered trademarks of TabTimer Pty Ltd © TabTimer Pty Ltd ABN: 99 137 415 948