

# VIBRATING 5-ALARM PILLBOX

## FEATURES

This pillbox helps you organize your daily pills, and reminds you with audio and/or vibrating alarm 5 times per day. No reset required. So you never forget to take your medicine again! Features include:

- Five alarms per day either audio or vibration or both. Snooze function.
- Key-lock function to prevent accidental trigger.
- Compact design for easy carry in pocket or purse.
- Large LCD display with LED backlight.
- Pulse meter for pulse rate monitor.

## INSTALLING AND REPLACING BATTERIES

Your Pill Box is powered by one CR2025 Lithium battery (preinstalled) for time function and one UM4 (AAA) battery for vibration, alarm sound and LED backlight.

Follow these steps to install the UM4 (AAA) battery.

1. Slide the **OPEN** switch in the direction of the arrow and lift the top body and separate it from the lower body.
2. Place one fresh UM4 (AAA) battery in the compartment as indicated by the polarity symbols (+ and -) marked in the compartment.
3. Place the top and lower bodies together and press them together until it clicks to lock.

Follow these steps to replace the CR2025 Lithium battery.




This battery is preinstalled, replace it when the LCD fades or the function is abnormal.

1. Slide the **OPEN** switch in the direction of the arrow and lift the top body and separate it from the lower body.
2. Slide holding bar to open in the direction of the arrow, then take it out.
3. Remove the old battery and install a new one with the positive side (+) up.
4. Replace the holding bar.

## SETTING THE TIME

1. Unlock the hold function by sliding down **HOLD**.
2. Press **T-SET** for 2 seconds and **T-SET** appears on the LCD, the hours digits flash. Rotate the Time Adjustment ring to set the correct hour.

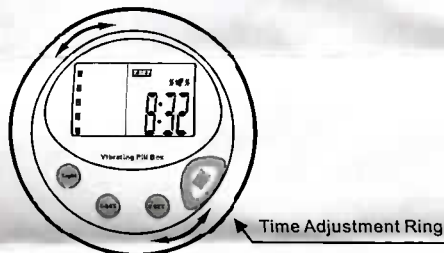

3. Press **T-SET** to confirm and the minutes digits flash, rotate the **Time Adjustment Ring** to set the correct minute.
4. Press **SET** again to return to the normal display mode.

**Note:** If no key is pressed for more than 30 seconds in Time Set mode, the unit will automatically return to normal time display mode.

## SETTING THE FIVE ALARMS

1. Unlock the hold function by sliding down **HOLD**.
2. Press **⏰** -**SET** for 2 seconds and **AL. SET** appears on the LCD, the first alarm time flashes. Rotate the **Time Adjustment Ring** to set the correct hour. Each click changes the time by 30 minutes.
3. To skip the alarm, set the time between PM 11:30 and 12:00, and " - : - " appears indicative of no alarm.
4. Press **⏰** -**SET** to confirm the alarm-**1** and the alarm-**0** time flashes, rotate the **Time Adjustment Ring** to set the correct time.
5. Repeat step 3 and 4 to set the alarm -**0**, alarm -**0**, and alarm -**0**.
6. Press **⏰** -**SET** to return to the normal time display mode.

At the preset alarm time, the alarm (sound or/and vibration) activates for 30 seconds while the corresponding alarm time flashes and "⏰" appears on the LCD.

- (a) If no button is pressed, the alarm automatically stops after 30 seconds then repeats the 30-second alarm twice every 5 minutes. Then the alarm time turns steady but " : " flashes, indicating the pill has not been taken.
- (b) If any button is pressed during any of the 30-second alarm, the alarm silences but the 5-minute snooze function remains.
- (c) If the pill compartment is rotated during the alarm or within 15 minutes before the alarm, the alarm time changes to " - : - " indicating that the pill had been taken.

## SELECTING THE AUDIO AND VIBRATING ALARM

There are three alarm modes to select from: **beeps** (🔊), **vibration** (📳) and **beep+vibration** (🔊📳). To select different alarm mode, slide the switch to **BEEP**, **VIB** or **BEEP+VIB**. In **BEEP+VIB** the alarm sound the beeps for 2 seconds followed by vibration for 2 seconds, repeating.

## RESETING THE FIVE ALARMS


All the 5 alarms automatically reset to their original alarm times at 0:13 A.M daily.

## USING THE PULSEMETER

A person's pulse rate indicates the number of times that the heart beats in one minute. Follow these steps to take pulse.

1. Press **PULSE START** to enter the pulse meter mode.
2. Place your index finger on the pulse sensor.
3. Maintain a light and even finger pressure on the sensor such that "♥" flashes steadily.
4. In 15-60 seconds, the unit sounds a beep and displays the pulse rate.
5. If no pulse is measured within 1 minute, the unit beeps once and returns to time mode. You can try again by pressing **PULSE START**.
6. To return to Normal mode, press **T-SET** or **⏰** -**SET**.

**Note:** To get an accurate reading, breathe normally and do not talk during measurement.

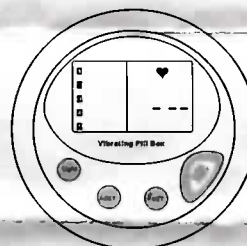

## FILLING THE PILL BOX

Fill the numbered compartments (1 to 5) with pills by rotating the cover to access each compartment.

## TURNING ON BACKLIGHT

Press **LIGHT** to turn on the backlight for 10 seconds.

## RESETING THE UNIT

When the pillbox shows abnormal function, such as "frozen display", "abnormal display", "frozen button" etc, you need to initialize the unit by pressing **RESET** using a pointed object such as paper clip. The **RESET** button is located in the inside of the top body.