


Instruction

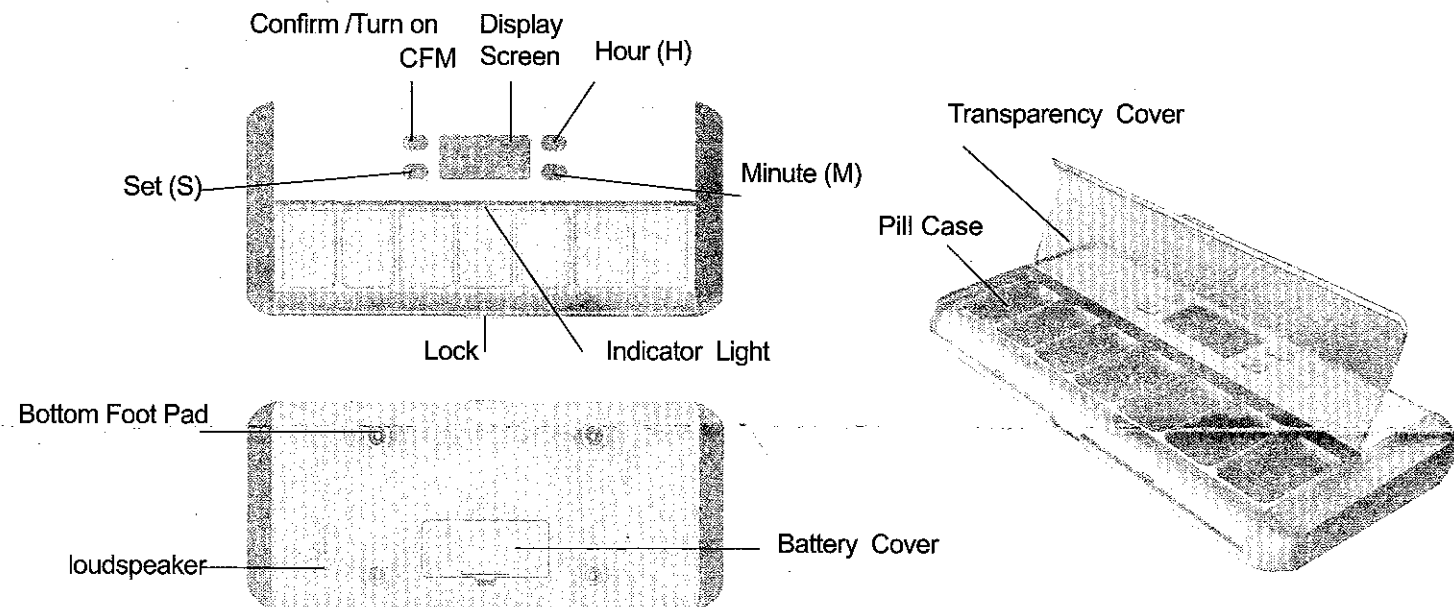

Power Supply

Use 2 units of No. 7 (AAA battery). Open the battery cover and load the battery correctly.

Power on / Shutdown / 12/24 hour conversion

1. The newly loaded battery will automatically start up and display in full screen.
2. Shutdown: long press CFM button for 3 seconds to shut down;
Power on: long press CFM button to start up in shutdown mode (the original alarm and time will be automatically remembered after power on).
3. 12/24 hour conversion: the battery is automatically loaded in the default mode of 24 hours.
The full screen will be displayed within 8 seconds after the power is connected.

Set the alarm

Press the S key 00:00 to number 1 will be flashing.

Press H button set Hours (long press quick adjustment), Press M button set Minuts (long press quick adjustment).

Press CFM-the keys to determine (After determine, the number with bell symbol not flashing)

Press the S key number 2 will be flashing, after confirm adjust time press CFM buttons.

Then press the S key set the third set of alarm, and so on, can be set up in total 7 set to remind the alarm. Turn off/turn on the alarm

Turn off/ Turn on alarm

Press the S key to the alarm that needs to be turned off, and press the H and M keys at the same time (the corresponding number flashing indicates that the alarm has been turned off successfully).

Press the S button to turn on the alarm, and press the CFM button to turn on the alarm.

Clock Settings

1. After loading the battery (within 10 seconds), press the M button to enter the clock mode, press the H button to adjust the hour (long press for quick adjustment), press the M button to adjust the minute (long press for quick adjustment), and press the CMF button to confirm.
2. Press S button 8 times to enter the clock mode, press H button to adjust the hour, press M button to adjust the minute, and press CFM button to confirm.

LED backlight

Press any button and the backlight will turn on. When the alarm goes off, the light will flash automatically.

Vibration reminder, light reminder and miss reminder

1. When the time is up, Not pressing any buttons alarm will ring for one minute and vibrate to remind (press any button to stop).
2. When the bell rings, a red light will turn on the pill box corresponding to the alarm, indicating the pill box, and it will turn off automatically after one minute of brightness (the red light will also turn on after turning off the bell, and it will turn off automatically after one minute).
3. Missed reminder: there is no button to operate when the bell rings. After one minute, there will be a short reminder bell and light every three minutes. Alarm ring and light, will continue to remind 15 times, press any button to stop.

Attention

1. When the power is switched on, it will ring and display the full screen for 8 seconds.
When no button is pressed for 8 seconds, it will automatically enter the clock mode. Press M to enter the time setting mode.
2. When setting the clock and alarm, press the CMF button to confirm the setting; otherwise, it will return to the time display after flashing for 5 seconds, and press the CMF button twice to return to the time display.
3. When setting the clock and alarm, set the hour and minutes and then press the CMF button, otherwise you will unintentionally set the wrong alarm (when the number flashes, press the CMF button to confirm the setting).
4. The set clock alarm clock will automatically remember the time whether it is on or off, and will only disappear when the power is disconnected (battery is taken out).
5. The maximum Angle of opening the transparent cover of the medicine box is 120°. Do not force the back plate to avoid damage.
6. If the alarm is set successfully, the number alarm will not flash. If the alarm is not set successfully, the symbol and number will flash.

Replace the battery

When the LED light is dimmed or the screen is dimmed, please open the battery cover on the back and replace 2 AAA (No. 7) batteries to ensure that the batteries are loaded in the correct direction.

Product size: 20*9*2CM Material: ABS+PCTG